




Distribuito in ITALIA da  
**Li StarFish S.r.l.**  
Via Cavour, 35  
20063 Cernusco S/N (MI)  
telefono 02-92150794  
fax 02-92157285  
info@listarfish.it  
www.listarfish.it

# IDK Extract<sup>®</sup>

*For sample preparation of stool samples to be used in ELISAs*

Gültig ab / Valid from 04.07.2014

REF K 6999

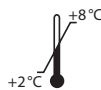

IVD CE

## 1. INTENDED USE

This extraction buffer concentrate *IDK Extract*<sup>®</sup> is intended for the sample preparation of stool samples to be used in the following ELISAs:

- Calprotectin (K 6927, K 6967)
- Pancreatic elastase (K 6915)
- Lactoferrin (K 6870)
- Hemoglobin (K 7836D)
- Hemo-haptoglobin complex (K 7837D)
- Anti-transglutaminase antibodies (K 9393)
- Anti-gliadin antibodies (K 9311)
- sIgA (K 8870, K 8880)
- $\alpha$ 1-antitrypsin (K 6750, K 6760)
- Albumin (K 6330)
- EDN (K 6811, K 6821)
- Lysozyme (K 6900, K 6901)
- b-Defensin 2 (K 6500)

For *in vitro* diagnostic use only.

## 2. MATERIAL SUPPLIED

Cat. No.	Label	Kit components	Quantity
K 6999	IDK Extract <sup>®</sup>	Stool sampel tube filled with IDK Extract <sup>®</sup>	100 units

## 3. STORAGE AND PREPARATION OF REAGENTS

- The **extraction buffer concentrate *IDK Extract*<sup>®</sup>** (EXBUF) must be diluted **according to the manual of the ELISA assay used**, mix well. Crystals could occur due to high salt concentration in the stock solutions. Before dilution, the crystals must be redissolved at 37°C in a water bath. The **extraction buffer concentrate *IDK Extract*<sup>®</sup>** is stable at **2 - 8 °C** until the expiry date stated on the label. Diluted **buffer solution** (extraction buffer) can be stored in a closed flask at **2 - 8 °C for three months**.

## 4. TECHNICAL HINTS

- Do not interchange different lot numbers of any kit component within the same assay.
- Reagents should not be used beyond the expiration date stated on kit label.
- Avoid foaming when mixing reagents.
- Do not mix plugs and caps from different reagents.
- The assay used should always be performed according the enclosed manual.

## 5. GENERAL NOTES ON THE TEST AND TEST PROCEDURE

- Warranty claims and complaints regarding deficiencies must be logged within 14 days after receipt of the product. The product should be send to us along with a written complaint.

### Used symbols:



Temperature limitation



Catalogue Number



For research use only



Contains sufficient for <n> tests



Manufacturer



Use by



Lot number